



# U.S. DIAGNOSTICS

QUALITY DETECTION PRODUCTS

[www.usdiagnostics.com](http://www.usdiagnostics.com)

## ProScreen™ Drugs of Abuse Dip Card



- Convenient, flood-proof “drop-in” design.
- Test for up to 12 drugs on a single device.
- Available with on-board adulteration testing.
- Negative Results in 2 minutes or less.

---

Made in the **USA**

---

# ProScreen™ Dip Instructions


## STEP 1

Have the donor provide a specimen in the Collection Cup and return with slotted lid in place.



## STEP 2

Verify the temperature is within range using the affixed temperature strip. (90-100°F)

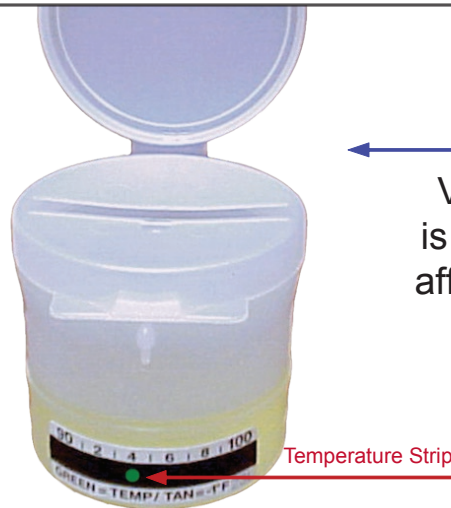

## STEP 3


Insert the card through the slotted lid in the collection cup.

## STEP 4

Test will run in 1-2 minutes. Read results when lines appear.




## Interpreting the Results

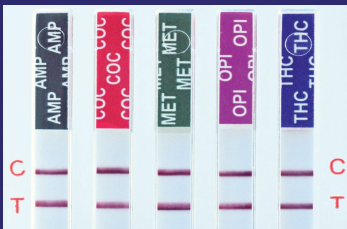


**NEGATIVE RESULT**  
A *Negative* test is indicated by the presence of a test line for each designated drug.



**NEGATIVE RESULTS**  
The presence of a very light "T" line no matter how faint, is a *Negative* test.

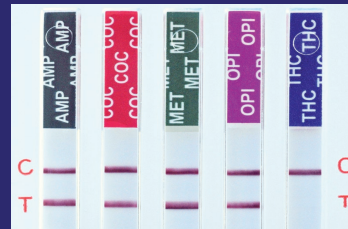

**POSITIVE RESULT**  
A *Positive* test is indicated by the Presence of a "C" line and the Absence of a "T" line. Wait 5 minutes to read Positive Results.

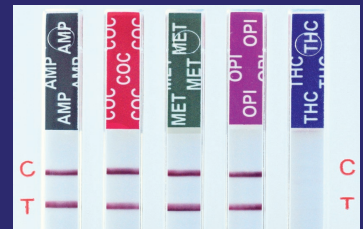

**INVALID RESULT**  
An *Invalid* test is indicated only when the "C" line is completely missing in one or more test windows. If this happens, run another test.